

Quick Reference - Topical Management - Pressure Wounds

MDS 3.0 v. 1.16 TERMINOLOGY OCTOBER 2018

Stage 1 Pressure Injury	Stage 2 Pressure Ulcer	Deep Tissue Injury <i>(skin intact)</i>	Shallow Stage 3 or Stage 4 Pressure Ulcer	Deep Stage 3 or Stage 4 Pressure Ulcer	Unstageable Pressure Ulcer <i>(slough / eschar)</i>
<p>© 2012 Gordian Medical, Inc.</p>	<p>© 2012 Gordian Medical, Inc.</p>	<p>© 2012 Gordian Medical, Inc.</p>	<p>© 2012 Gordian Medical, Inc.</p>	<p>© 2012 Gordian Medical, Inc.</p>	<p>© 2012 Gordian Medical, Inc.</p>
<p>Protect Remove cause Promote Healing</p>	<p>Protect Manage Exudate Promote Healing</p>	<p>Protect Remove cause Promote Healing</p>	<p>Protect Manage Exudate Promote Healing</p>	<p>Protect Manage Exudate Fill dead space if necessary Promote Healing</p>	<p>Protect Manage Exudate Fill dead space if necessary Promote Healing</p>
<p>ALL WOUNDS: ✓ <u>Cleanse</u> wound and periwound skin thoroughly ✓ <u>Protect</u> periwound skin with skin prep ✓ <u>Dressing</u> package has detailed instructions</p>					
<p>Area exposed to friction: Skin Prep</p> <ul style="list-style-type: none"> • DermaPrep • StingFree <p>Transparent Film</p> <ul style="list-style-type: none"> • DermaView • DermaView II <p>Area exposed to urinary or fecal incontinence:</p> <p>Skin Protectants*</p> <ul style="list-style-type: none"> • PeriGuard • Renew PeriProtect • Renew Dimethicone Skin Protectant • Lantiseptic Dry Skin Therapy (30% lanolin) • Lantiseptic Moisture Shield (50% lanolin) • DermaSeptin • DermaMed <p>*Refer to Skin Care Quick Reference for details</p>	<p>Drainage: none to minimal</p> <p>Transparent Film</p> <ul style="list-style-type: none"> • DermaView • DermaView II <p>Hydrocolloid</p> <ul style="list-style-type: none"> • DermaFilm <p>Hydrogel</p> <ul style="list-style-type: none"> • DermaSyn Gel • DermaGauze • AquaDerm <p>Impregnated Gauze</p> <ul style="list-style-type: none"> • Xeroform Gauze • Petrolatum Gauze • Oil Emulsion Gauze <p>Drainage: moderate</p> <ul style="list-style-type: none"> • ComfortFoam Border Lite • DermaLevin <p>Drainage: heavy*</p> <ul style="list-style-type: none"> • Bordered Foam • DermaFoam • ComfortFoam Border <p>*not typical for a Stage 2</p>	<p>Intact skin: Skin Prep</p> <ul style="list-style-type: none"> • DermaPrep • StingFree <p>*If a Deep Tissue Injury opens, select a dressing that is appropriate to manage the wound characteristics seen on assessment:</p> <ul style="list-style-type: none"> • Depth of open area • Wound bed appearance • Amount of drainage • Amount of bioburden <p><small>Content provided for educational/informational purposes only and believed to reflect best practice at publication time. Please consult a medical or legal professional for medical or legal advice. No legal liability or other responsibility is accepted by or on behalf of DermaRite for any errors, omissions or statements within. ©2019 DermaRite Industries LLC. www.dermarite.com All rights reserved.</small></p>	<p>Drainage: none to minimal</p> <p>Transparent Film</p> <ul style="list-style-type: none"> • DermaView/DermaView II <p>Hydrocolloid</p> <ul style="list-style-type: none"> • DermaFilm <p>Hydrogel</p> <ul style="list-style-type: none"> • DermaSyn/DermaGauze <p>Drainage: moderate</p> <ul style="list-style-type: none"> • ComfortFoam Border Lite • DermaLevin <p>Drainage: heavy*</p> <ul style="list-style-type: none"> • Bordered Foam • DermaFoam • ComfortFoam • ComfortFoam Border <p>Antimicrobial Options</p> <ul style="list-style-type: none"> • DermaSyn/Ag • SilverDerm 7 • ComfortFoam/Ag • ComfortFoam Ag Border • DermaBlue+Foam Transfer • DermaBlue+Foam <p>Fragile Wound Bed</p> <ul style="list-style-type: none"> • ComfiTel <p>Stalled Wound Healing</p> <ul style="list-style-type: none"> • SilvaKollagen Gel • DermaCol • DermaCol/Ag 	<p>Drainage: none to minimal</p> <p>Hydrogel</p> <ul style="list-style-type: none"> • DermaGauze <p>Drainage: moderate /heavy</p> <p>Alginate</p> <ul style="list-style-type: none"> • DermaGinate <p>Gelling Fiber</p> <ul style="list-style-type: none"> • AquaRite Extra CMC <p>Superabsorbent</p> <ul style="list-style-type: none"> • HydraLock SA <p>Non-bordered Foam</p> <ul style="list-style-type: none"> • DermaFoam • HydraFoam <p>Antimicrobial Options</p> <ul style="list-style-type: none"> • DermaSyn/Ag • SilverDerm 7 • SilvaKollagen Gel • DermaGinate Ag • DermaBlue+Foam Transfer • DermaBlue+Foam <p>Fragile Wound Bed</p> <ul style="list-style-type: none"> • ComfiTel <p>Stalled Wound Healing</p> <ul style="list-style-type: none"> • DermaCol 100 (powder or Microfibrillar sheet) • DermaCol • DermaCol/Ag 	<p>Stable/intact eschar on heel</p> <ul style="list-style-type: none"> • StingFree or DermaPrep <p>Unstable eschar on heel</p> <ul style="list-style-type: none"> • Evaluate vascular status <p>Adequate blood flow needed for autolytic debridement</p> <p>Minimal Drainage</p> <ul style="list-style-type: none"> • DermaSyn • DermaGauze • DermaFilm • DermaView • DermaView II <p>Drainage: moderate /heavy</p> <p>Alginate</p> <ul style="list-style-type: none"> • DermaGinate <p>Gelling Fiber</p> <ul style="list-style-type: none"> • AquaRite Extra CMC <p>Superabsorbent</p> <ul style="list-style-type: none"> • HydraLock SA <p>Non-bordered Foam</p> <ul style="list-style-type: none"> • DermaFoam/HydraFoam <p>Antimicrobial Options</p> <ul style="list-style-type: none"> • DermaSyn/Ag • SilvaKollagen Gel • DermaGinate Ag • DermaBlue+Foam Transfer • DermaBlue+Foam
<p>CHANGE THE DRESSING: ✓ If the sealed border is breached ✓ If strike-through drainage or contamination is noted ✓ As directed by a qualified healthcare professional</p>					

Wound Care Products on Formulary

DermaRite®

Category	Functions	DermaRite Products in Category
Calcium Alginate	Absorption, moist wound healing, autolytic debridement, hemostasis <i>Typically used as a primary dressing for a wound with moderate to heavy drainage</i>	DermaGinate <i>Silver version:</i> DermaGinate/Ag
Collagen	Addresses enzyme imbalance in stalled chronic wound and supports formation of granulation tissue <i>Used as a primary dressing</i>	DermaCol Collagen Matrix, DermaCol 100 Collagen Powder Wound Filler, DermaCol 100 Sheet <i>Silver version:</i> DermaCol/Ag, SilvaKollagen Gel
CMC* (Gelling Fiber Dressing)	Moist wound healing, autolytic debridement, highly absorbent. <i>Typically a primary dressing</i> <i>* often referred to as a "hydrofiber"</i>	AquaRite Extra CMC
Composite	Absorption, protection, securement of primary dressing <i>Can be used as a primary or secondary (cover) dressing</i>	DermaDress, DermaView II Island
Compression	Management of chronic venous insufficiency, venous ulcers <i>Use as directed by physician order</i>	FlexPress4 (4-layer graduated compression wrap) , UnnaRite w/Zinc, UnnaRite Zinc+ calamine (paste bandages)
Contact Layer	Protects wound bed from trauma associated with dressing changes <i>Use as a primary dressing</i>	ComfiTel (silicone contact layer), <i>Silver Contact Layer:</i> SilverDerm7
Foam	Absorption, moist wound healing, autolytic debridement, supports normothermia Antimicrobial version indicated for wounds needing broad spectrum antimicrobial protection within the dressing <i>For wound with moderate to heavy drainage. Can be a primary or secondary dressing</i>	Bordered Foam, DermaFoam, DermaLevin, HydraFoam <i>Antimicrobial version:</i> DermaBlue+ Foam (Methylene Blue +Gentian Violet +silver), DermaBlue+ Foam Transfer
Foam - silicone	Absorption, wound bed protection, moist wound healing, autolytic debridement, supports normothermia	ComfortFoam Border Lite, ComfortFoam Border, ComfortFoam <i>Silver version:</i> ComfortFoam Ag, ComfortFoam Ag Border
Gauze	Absorption, protection, securement of primary dressing <i>Can be a primary or secondary dressing</i>	Bordered Gauze, Sterile Bordered Gauze
Gauze-impregnated	Non-adherent, provide moist wound environment <i>Indications for use vary based on compound</i>	DermaGauze Hydrogel-Impregnated Gauze, DermaRite: Oil Emulsion, Petrolatum and Xeroform Gauze
Hydrocolloid	Moist wound healing, autolytic debridement <i>Typically used as a primary dressing</i>	DermaFilm: Thin with Border, X-Thin Clear with Grid, HD
Hydrogel	Hydration for dry wounds, autolytic debridement <i>Use as a primary dressing</i>	AquaDerm Hydrogel Sheet, DermaSyn, DermaGauze <i>Silver version:</i> DermaSyn/Ag
Skin Prep	Moisture barrier, protection of fragile periwound skin	StingFree, DermaPrep
Closure Strips	Wound closure	StayStrips
Super Absorbent	Multi-layer, highly absorbent, non-adherent with gelling core. <i>Primary dressing for heavily draining wound</i>	HydraLock SA
Tape	Dressing and device securement	RiteFix Tape (breathable, non-woven), ComfiTape Silicone Tape (waterproof)
Transparent Film	Moist wound healing, autolytic debridement, bacterial barrier <i>Typically used as a primary dressing</i>	DermaView, DermaView II, DermaView IV
Wound Cleanser	Wound bed preparation	DermaKlenz, SafeWash